

Calibration Log

Date Used	Recalibration Date	Notes

SEE OWNER'S MANUAL FOR RE-CALIBRATION INSTRUCTIONS THAT MUST BE PERFORMED DEPENDING ON FREQUENCY AND METHOD OF USAGE.

AlcoHAWK® Elite Quick Guide

Be sure to read the entire owner's manual and follow procedures carefully to obtain optimal results.

Operating Instructions

WAIT 20 MINUTES AFTER SMOKING, EATING, OR DRINKING BEFORE USE. FAILURE TO DO SO CAN ALTER THE BAC READING AND CAN DAMAGE THE SENSOR.

- 1) Insert a new mouthpiece each time you perform a test.
- 2) If it has been more than 24 hours since the device was last used, prime the sensor by blowing into the mouthpiece for 4 to 5 seconds before turning on the device.
- 3) **PRESS** the **Power On/Off** button. A single beep will sound and the unit will display the temperature in Fahrenheit (F).

Note: If the unit reads a temperature below 50 F, "**Cool**" appears on the LCD display to warn the user that the temperature is lower than the suggested testing range. If the temperature is above 104 F, "**Hot**" appears on the LCD display to warn the user that the temperature at the time of testing is higher than the suggested testing range.

- 4) After the temperature scan, **WAIT** until the **COUNTDOWN IS COMPLETE** and you hear a **SINGLE BEEP**. The "**RDY**" will appear when the device is ready for use.

Note: The unit will restart the countdown at 10 if the sensor is not ready after the original countdown from 60.

Note: The device will not shut off until a test is performed OR until 15 seconds of non use elapses.

- 5) Take a **DEEP BREATH AND BLOW STEADILY** (for at least 5 seconds) until a **BEEP** sounds to signal completion of the testing.

Note: The unit will display a 0 after approximately 1 – 2 seconds if a trace level of alcohol is not detected in the breath sample (i.e., at least 0.001% BAC).

- 6) **SEE OWNERS MANUAL FOR HELP UNDERSTANDING THE RESULTS, ERROR MESSAGES AND PRECAUTIONS**

- 7) For more testing, wait one (1) minute and repeat steps 1 – 4.

THIS DEVICE IS INTENDED TO MEASURE ALCOHOL IN THE HUMAN BREATH, SEE PRECAUTIONS AND WARNINGS BEFORE USE.

DO NOT DRINK AND DRIVE.

SEE OWNER'S MANUAL FOR RE-CALIBRATION INSTRUCTIONS THAT MUST BE PERFORMED DEPENDING ON FREQUENCY AND METHOD OF USAGE.

