

AlcoHAWK® One Test

Disposable Breath Alcohol Tester

Traditional digital breath alcohol testers are a great investment for buyers looking to test several times in a month or multiple times in one day. The AlcoHAWK® One Test is a more cost-effective option for individuals testing themselves or loved ones just a few times or less every month. Plus, with the cost savings, you can order more than one AlcoHAWK® One Test and share a pack with friends and/or family members!



Easy to Use: A red dot on the tube will change to a white dot letting the user know when a sufficient breath sample has been obtained. Breath sample integrity is critical in breath alcohol testing - this simple color change is easy for users to understand.

A black line also helps to ensure users understand the test results. When the crystals over the testing level line have changed color, it is indicative of a positive test result (the user's blood alcohol content is the applicable testing level or higher). Similar products do not use this method, leaving the users unsure of how many of the crystals need to change color to indicate a positive result or whether the color has sufficiently changed.

Affordable: A cost-effective alternative to a digital screener. Purchase the One Test in bulk to keep in multiple locations and test quickly at a fraction of the cost.

Packaging: The One Test is available for purchase in individually wrapped units or bulk. Point of purchase displays are also available.

Compact Design: The small tube design is very discreet and easy to store.

Available Testing Levels

.05% BAC

.08% BAC

Affordable

Effective

Fast

AlcoHAWK®

Q3 Innovations www.q3i.com
2349 Jamestown Avenue, Suite 4, Independence, Iowa 50644
Tel: 319-334-3412 Fax: 319-334-3421 Email: sales@q3i.com

AlcoHAWK[®] One Test



Blow



Stop Blowing



Under Testing Level



Over Testing Level

How Does It Work?

The AlcoHAWK[®] One Test contains chemical reacting crystals that react to alcohol. When you blow into the tube, the alcohol in your breath reacts with the crystals. As more alcohol is detected, more crystals turn green in color. If the green color reaches or exceeds the black line, you are at or above the testing level.

Specifications

Dimensions (L*W*H)	4" x .5" x .5" (10.16 cm x 1.27 cm x 1.27 cm)
Weight	0.2 oz (5.7 g)
Housing	Clear Glass Tube with Plastic Caps
Sensor	Alcohol Reactive Chemical Crystals
Blowing Time	12 Seconds
Response Time	120 Seconds ¹
B.A.C. Range	.05% or .08% BAC (Blood Alcohol Concentration) ²
Accuracy	± 2% Error Margin
Use	Single Use
Shelf Life	18 Months

1 - Estimate, results may vary

2 - B.A.C. Range depends on testing level of unit purchased

3 - Read Instruction Guide before use and follow instructions carefully