

AlcoHAWK[®] One Test

Disposable Breath Alcohol Test

EXP DATE:
11-23-2012

Fast • Effective • Affordable

LOT#:
2331

Read Instructions Before Use • Keep Out of Reach of Children

Intended Use: This device is an *in vitro* medical device to qualitatively detect the presence of alcohol in the human breath. It is a disposable screening device for one-time use. The color of the crystals will change if alcohol is detected. This device is to be used only as a screening device and is only an indication of the possible presence of alcohol in the blood of the test subject.




Made in Canada



Distributed By Q3 Innovations
2349 Jamestown Avenue, Suite #4
Independence, Iowa 50644
888-399-1687
www.q3i.com

TEST LEVEL: 0.05% BAC

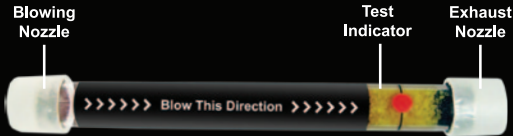

DO NOT DRINK ALCOHOL AND DRIVE

**ALWAYS USE A DESIGNATED DRIVER
AFTER DRINKING ALCOHOL**

BE SAFE AND DRINK RESPONSIBLY



Wait **20 minutes** after you smoke, eat or drink before testing. Failure to wait 20 minutes can give wrong BAC readings.

Operation

- Step 1)** Press firmly on both plastic caps at each end of the tube to pierce the aluminum.
- Step 2)** Wrap your lips around the nozzle on the end opposite of the crystals. Blow slowly and constantly for about 12 seconds. Do not blow through the end that has a red dot.
- Step 3)** Stop blowing as soon as the red dot becomes white.
- Step 4)** Wait until the white dot turns back to red.
- Step 5)** Read the results.

Understanding the Results

When alcohol comes in contact with the yellow crystals the crystals will turn green. More crystals turn green as more alcohol is detected.

You are over 0.05% BAC if the green color reaches or passes the black line.

ATTENTION: In some cases, it may take up to 60 minutes to reach the highest blood alcohol concentration (BAC) after the last consumption of alcohol. Your BAC can increase up to 0.025% BAC.

Disclaimer

The result of this test cannot in any case be opposable to the results of a test taken by an official. The use of this test and the decisions of the user based on the result of the test are his/her sole responsibilities. The manufacturer, distributors or retailers are not responsible for the user's actions. This device is designed to provide a suggestion of your blood alcohol concentration only at the time the test is taken. Each individual body responds uniquely to the use of alcohol. A person's ability to drive may become legally impaired at a point below the legal BAC limit.

Precautions

This product is made with glass. It is fragile. Do not use the test if the aluminum parts (nozzles) are already broken. Do not use the test if the glass is cracked or broken. Do not use the test if the crystals are already green. Do not ingest. In case of contact with the eyes or skin, rinse under water. Please dispose properly after use. Avoid storing over 113°F (45 °C) and extended exposure to light. Do not remove plastic ends. Manufacturer, distributors and retailers are not responsible for use or misuse of this product for any reason. Tester results should not be read by readers who are color-blind or visually impaired. Results are best viewed under white light. It may be hard to read Tester results under certain streetlights or under dim lighting. Do not use Tester after expiration date.



Blue

Stop
Blowing

Under
0.05% BAC

Over
0.05% BAC