

AlcoHAWK® Slim 2 Quick Guide

v061808



Be sure to read the entire owner's manual and follow procedures carefully to obtain optimal results.

Operating Instructions

Wait **20 minutes** after the subject smokes, eats or drinks before testing. Failure to wait 20 minutes can give inaccurate BAC readings and harm the sensor.

- Step 1)** Place a fitted cover over the mouthpiece each time you test another individual.
- Step 2)** If it has been more than 24 hours since the device was last used, **prime the sensor** by blowing into the mouthpiece for 4 to 5 seconds **before** turning on the device.
- Step 3)** **PRESS** the red **Power On/Off** button and **HOLD** for **one (1) second**. A single beep will sound and the device will display the room temperature and then count down from 60 as it prepares the sensor for testing.
Note: The device will restart a count down from 10 if the sensor is not ready within the first countdown from 60.
- Step 4)** **WAIT** until the unit displays "Rdy", "BLOW" and you hear a **SINGLE BEEP**.
Note: The device will shut off after 15 seconds of non use.
- Step 5)** Take a **DEEP BREATH AND BLOW STEADILY** (for at least 5 seconds) until a **BEEP** sounds to signal completion of the testing.

THIS DEVICE IS INTENDED TO MEASURE ALCOHOL IN THE HUMAN BREATH.

DO NOT DRINK AND DRIVE.

SEE OWNER'S MANUAL FOR RE-CALIBRATION INSTRUCTIONS THAT MUST BE ROUTINELY PERFORMED.

